

Hollister 9650 Sterile Night Drainage Bag

Instructions for Use

Connecting to catheters and sheaths

1. Wash hands. Use sterile technique if attaching to catheter.
2. Attach inlet adaptor to catheter, sheath or leg bag.
3. Attach bag to hanger for drainage.
4. To empty bag, flip tap open. Snap closed to secure.
5. Replace tap to closed position when empty.

Maintenance with Catheters

- Leave the bag attached to catheter to maintain a "closed" system.
- Change the bag weekly or as per clinician's instructions, using a sterile technique.

Decontaminant Solution Preparation (for sheath / leg bag drainage)

- Option 1:** Household detergent solution.
Dilute 5ml detergent in 150ml of warm tap water.
- Option 2:** Weak vinegar solution (for odour removal). Dilute 30ml vinegar in 120ml of cold tap water.
- Option 3:** Milton's solution



Product Code: 9650

Cleaning the Night Drainage Bag (for sheath / leg bag drainage)

1. Remove bag daily.
2. Empty all urine from bag into toilet.
3. Close outlet tap, fill bag with cold tap water and shake vigorously.
4. Empty water through drainage outlet into toilet and repeat steps 3 and 4.
5. Refill bag with decontaminant solution and agitate for 30 seconds.
6. Drain solution into toilet, open outlet and hang bag in shaded place to air dry.
7. All parts must be completely dry before the next use. Replace bag weekly or at signs of wear and tear.

For more information or samples please call Customer Engagement on

Australia Freecall: 1800 219 179
New Zealand Freecall: 0800 678 669